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ENGLISH EUA

PRODUCT DESCRIPTION

Column Instrument Kit - Biomecânica: Instruments are made of stainless steel 304 and 420, plastics (teflon), polyacetal, polyethylene, aluminum. Instrument finishing is performed by mechanical polishing followed by glass microsphere sandblasting in case of aluminum, steel grit sandblasting to metal instruments. They are divided in:

TYPE	DESCRIPTION	USE
Cut	Saw, male, osteotome	They are used to prepare gauges or implants. Sharp parts are sharpened and polished, the other parts are polished or sandblasted. They are supplied non-sterile.
Tightening	Wrench	They are used to place or remove screws and nuts. Their finishing can be polished or sandblasted. Wrench handle can be made of aluminum or steel. They are supplied non-sterile.
Gauges	Positioners	They are used in medical device positioning and placement, in cut and hole marking. Their finishing can be polished or sandblasted. They are supplied non-sterile.
Drilling	Drill bit and milling cutter	They are used in bone drilling and milling, that enable male input to screw, wire and rod fastening. Their finishing is polished. They are supplied non-sterile.
Impaction / Extraction	Hammer, impactor, hooks	They are used to place and remove orthopedic devices. Their finishing is performed through sandblasting. They are supplied non-sterile.
Milling	Scrapers	They are used in bone segment preparation to place orthopedic device. Their finishing is polished and sandblasted. They are supplied non-sterile.
Extension	Extensions	Used as intermediate instrument. Their finishing is polished and sandblasted. They are supplied non-sterile.
Box	Storage boxes	They are drilled externally, so that the steam saturated under pressure (autoclave) can penetrate all box internal space. Inside it, there are stainless steel trays with specific fittings to place instruments. Brushed material is used to manufacture 1.2 mm thick boxes and trays. Boxes and trays, after they are ready, undergo electrolysis bath, leaving for 5 minutes inside the tank. To assemble boxes and trays, stainless steel rivets are used.

PRESENTATION FORM

Instrument
1 unit
Instruments are stored in plastic envelopes with non-sterile label and external labeling. Metal trays are protected with bubble wrap and stored in metal box, designed to steam sterilization (autoclave). Metal box is protected with bubble wrap and labeled externally and identified as non-sterile.

INDICATIONS

Column Instrument Kit - Biomecânica is indicated as critical assistance to prepare the bone

segment to receive the column implant.

ANCILLARY COMPONENTS

Column Instrument Kit - Biomecânica does not have ancillary components.

ACCESSORIES

Column Instrument Kit - Biomecânica does not have any accessory with the purpose of integrating the medical product, granting to this product a complementary technical function or characteristic.

CONTRAINDICATIONS

Contraindication is connected to implant to be used, as provided in package inserts supplied with implants. Instruments should not be used for different purposes than intended.

WARNINGS

As Column Instrument Kit - Biomecânica is reused in another patient, it is required that it is cleaned, sanitized and then re-sterilized with steam in autoclave. This process has to be performed every time in case of use in new surgery. The reuse without undergoing sanitization and re-sterilization process is expressly forbidden.

PRECAUTIONS

Do not use this product if it is damaged.

Keep the in ventilated, dry place, away from light and severe weather action.

The surgeon must be familiar and have enough knowledge of column implant fastening surgical technique and its limitation, including before and after surgery, surgery technique adopted, precautions and potential risks.

As they are cutting, gauge and other instruments, the handling of this kit must be performed with caution in the following circumstances: in surgical act, in cleaning and sterilization handling, as well their transport, and they have to be store in relevant places identified in trays. To enable adequate deployment, column instruments manufactured by Biomecânica must be used, as they have been manufactured and designed specifically to be used with Biomecânica brand column implants. Using different manufacturer instruments may compromise the surgery. Through time, instruments tend to wear naturally due to regular use. Instruments should only be used as specified.

ADVERSE EFFECTS

There adverse effect reports.

INSTRUCTIONS FOR USE:

Surgery techniques vary according to surgeon knowledge and choice. The surgeon is responsible for the final choice of method, type and dimension of instruments and implants to be used, as well as after-surgery result evaluation criteria.

Column Instrument Kit - Biomecânica should be used during the surgical procedure, according to purpose each instrument has been developed. Such instrument used is indicated to be used by professionals only. Surgeons that manage such product use need to know perfectly implant processes, and column implant instrument and component handling.

MEDICAL PRODUCT STORAGE AND TRANSPORT SPECIAL CAUTIONS

- Do not use this product if it is damaged.

Column Instrument Kit - Biomecânica must be handled with care, preventing damage that can impact material quality and patient safety.

- Keep the in ventilated, dry place, away from light and severe weather action.

- As it is transported, bumps, vibrations and high temperatures (above 45°) must be prevented, as well as inadequate stacking.

* Cautions during transport are described in the package.

STERILIZATION

Column Instrument Kit - Biomecânica does is supplied non-sterile. They have to be sterilized before use. We recommend autoclave steam sterilization at the hospital (*ISO 11134: 1994 Sterilization of health care products - Requirements for validation and routine control - Industrial moist heat sterilization*).

Autoclave

It is a piece of equipment for sterilization through saturated steam under pressure.

Operating Instructions

For equipment start, check:

- if the circuit breaker is on;
- if the water supply valve is open;
- if the discharge valve is closed.

- 1 - open the equipment door;
- 2 - store the material to be sterilized properly
- 3 - close the equipment door;
- 4 - choose the cycle wanted according to the material to be sterilized;
- 5 - turn on the general switch;
- 6 - the cycle will be executed automatically, in sequence;

7 - as 'end of cycle' lamp is on, open the door partially for ten (10) minutes, approximately, for material chilling.

FOR FURTHER CLARIFICATION, REFER TO INSTRUCTION MANUAL SUPPLIED WITH AUTOCLAVE.

Another sterilization method: However, the following sterilization method apart from autoclave can be used, as defined by hospital institution:

- Ethylene Oxide Sterilization (E.T.O.) parameters and procedures provided in validation protocol and *EN550:1994 Sterilization of Medical Devices - Validation and Routine Control of Ethylene Oxide Sterilization*.

NOTE

The hospital institution is responsible for sterilization method, equipment, controls, and sterilization instructions used.

CLEANING AND DECONTAMINATION RECOMMENDED

General recommendations

After surgery and before sterilization, Column Instrument Kit - Biomecânica must be decontaminated using wide spectrum bactericide and fungicide solution, avoiding aggressive cleaning agents, metal brushes, so that instrument quality is not damaged.

RE-STERILIZATION

In case of re-sterilization, all cautions abovementioned must be followed.

REMARK

This product must be handled by qualified people.

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