**POSITIONER PENGO**

## PORTUGUÊS

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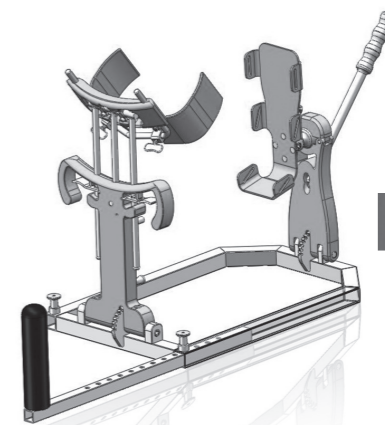
## ENGLISH

### POSITIONING SPECIFICATION

Biomechanica's **PENGO POSITIONER** is a simple and flexible distraction system which renders surgical procedures simpler, easier and faster. As compared to the orthopedic stress table, **PENGO POSITIONER** simplifies positioning of the patient and reduces requirements of the operating room team. **PENGO POSITIONER** is accessible in such way that the surgical procedures may be carried out in an ambulatory environment.

### PRODUCT DESCRIPTION

Biomechanica's **PENGO POSITIONER** is a system designed to assist the surgical procedures of the waist, femur, tibia and knee. The system is coupled at the supine or side position to the trails of most surgical tables. The system is comprised of the following sections:

#### 1) POS0001A - SUPPORTING BASE

Positioning of the supporting base shall be in supine and supported on a surgical table. The entire **PENGO POSITIONER'S** tensile system is on the base.

#### 2) POS0001B - STABILIZATION SUPPORT

Used to assist tension and stabilize the patient's limb. The support edge contains a patient's groin-rest.

#### 3) POS0001C - ARTICULAR SUPPORT

It is based on the rear section of the knee, preventing it from falling during flexion. It prevents the operating room team from being required to hold the knee in position for long periods of time.

#### 4) POS0001D - FOOT SUPPORT

The fractured leg is supported on the metallic rest according to the position informed by the surgeon and lock the system allowing for full stabilization. It prevents the operating room team from having to hold the foot pulled in position for long periods of time.

#### 5) POS0001E - TENSION LEVER

Used to cause system tension both in foot support and in the articular support.

### COMPOSITION

| CÓDIGO   | DESCRIÇÃO             | MATERIAL   |
|----------|-----------------------|--|
| POS0001  | PENGO POSITIONER      | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Stainless steel AISI 304 |
| POS0001A | SUPPORT BASE          | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Carbon fiber             |
| POS0001B | STABILIZATION SUPPORT | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Polyacetal Silicone      |
| POS0001C | ARTICULAR SUPPORT     | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Carbon fiber             |

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| CODE     | DESCRIPTION   | MATERIAL   |
|----------|---------------|--|
| POS0001D | FOOT SUPPORT  | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Carbon fiber<br>Vinyl<br>Neoprene Rubber<br>Velcro<br>Silicone |
| POS0001E | TENSION LEVER | Aluminum 6061-T-6<br>Aluminum 6262<br>Aluminum 7075<br>Stainless steel 303<br>Stainless steel 18-8<br>Stainless steel 17-4<br>Stainless steel AISI 304                                       |

#### INDICATION OF USE

**PENGO POSITIONER** System is intended to apply operating tension strength during operating procedures of waist, femur, tibia and knee.

#### WARNINGS

- The surgeon should be familiarized with the suitable techniques prior to using this device.
- Carefully read these instructions prior to using the device.
- Ensure that the operating table has the capacity for a patient with approximately 180 Kg.
- Do not exceed the threshold weight of 180 Kg for the **PENGO POSITIONER** System's patient.
- Any adjustment to the stabilization support, articular support or foot support at any time, shall change the limb tension. Significant changes to the tension may cause injury to the patient.
- So as to keep flexion and avoid sliding, ensure that the patient's heel is firmly sat to the foot support and his/her knee is firmly supported on the articular support, the support is intended to stabilize such rests.
- Remove all tools from the joint space prior to adjusting the limb tension.
- Do not adjust the distractor shaft when the ball joint of the foot stand is blocked, since the result may be injury to the patient.

#### PRECAUTIONS

- Prior to each use, inspect the device so as to ensure that it is properly operational and not damaged. Do not use a damaged device.
- Be careful in areas where fluid migration may occur. In particular, ensure that there is no fluid migration under the locking activation triggers or in the foot and knee locking systems.

#### COUNTER-INDICATION

This product is counter-indicated for patients without knee and ankle joints.

#### PACKAGING REMOVAL AND GENERAL INSPECTION

Carefully unpack and inspect all components sent with **PENGO POSITIONER**. If any item is damaged or missing, contact a BIOMECANICA representative.

Store the carton and packaging materials in case any component needs to be returned for repair.

#### STORAGE

Store in a fresh, safe and dry place.

#### DISPOSAL

As required, product disposal should follow the technical healthcare service waste management regulation, as described in Resolution RDC 306 dated 12/07/2004.

#### LEGAL WARRANTY TERM

(In accordance with the Consumer Protection and Defense Code: Act 8.078, dated September 11, 1990).  
The company BIOMECANICA, in compliance with Art. 26 of the Act 8.078, dated September 11, 1990, hereby ensures the consumer right to complain of the apparent or easily verifiable flaws in all products imported and traded by it, for 90 days, counted as of effective delivery of the products. In case of concealed flaw, the term begins at the time the flaw is evidenced, as provided for in Paragraph 3 of Art. 26 of the Act 8.078.

**For this Legal Warranty Term to be effective, the consumer shall abide by the following conditions:**

- Do not allow for unauthorized and disqualified individuals to carry out the procedure with the product concerned.
- Do not allow for undue use as well as misuse of the product concerned.
- Strictly follow the usage instructions in the Medical Usage Manual.

#### PRODUCT PRESENTATION FORM

##### Package Content

(1) One unit of **PENGO POSITIONER** (Code: POS0001), (1) One unit of Instructions of Use.  
The parts of **PENGO POSITIONER** may be separately purchased by using the codes below:

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